

INTEGRATED HEADREST

CATALOG NO. F-422PLH-2

USE THIS MANUAL FOR ALL AQUA CREEK PRODUCTS' ACCESS LIFTS

MANDATORY

LEAVE THIS MANUAL WITH OWNER

Read and follow all instructions.

Lift safety can only be ensured if the headrest is installed and operated according to these instructions.


- FOR SAFETY REASONS
- NEVER swim alone
- NEVER use the lift with a dry pool
- NEVER apply direct water pressure to the electronics
- NEVER permit children to play on or around the lift
- DO NOT allow children to use the lift without adult supervision



Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.


Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

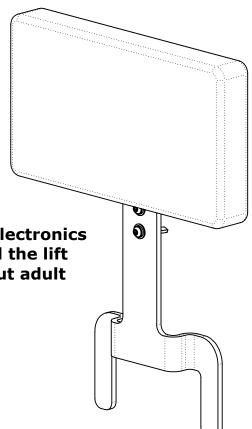

Aqua Creek Products, LLC 9889 Garrymore Lane Missoula, MT 59808 Toll Free: (888) 687-3552 Local/Intnl: (406) 549-0769 www.aquacreek.com



Aqua Creek Products PRODUCT COMPONENTS

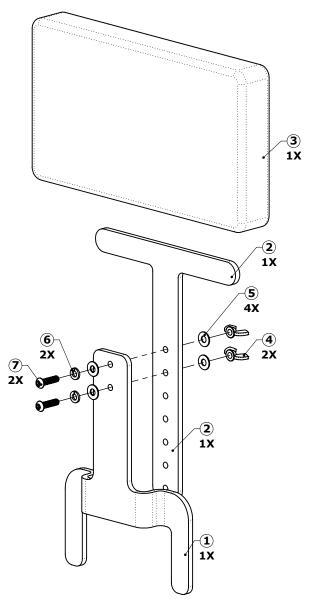

ASSEMBLY - HEADREST - INV01455				
ITEM	PART NO.	QTY.	REV.	DESCRIPTION
1	INV00854	1	P1	LASER/WJ - HEADREST SEAT BRACKET
2	INV00855	1	P1	LASER/WJ - HEADREST ADJUSTMENT BRACKET
3	INV00806	1	P1	HEADREST PAD
4	INV01181	2	N/A	NW - 1/4"-20 WING NUT
5	INV01170	4	N/A	WF - 1/4" FLAT WASHER
6	INV01182	2	N/A	WL - 1/4" LOCK WASHER
7	INV00341	2	N/A	BOLT - BHSCS 1/4"-20x1"



Aqua Creek Products PRODUCT ASSEMBLY

- 1. If the headrest pad has not yet been installed, install it first. The pad height may be adjusted now, or afte the assembly has been attached to the chair. Assemble fully as shown in **FIGURE 01.**
- 2. Slide the support bracket into the slots on top of the seat. FIGURE 02

The Integrated Headrest assembly is now ready to adjust and use.

